

PRIOR TO ENTERING ROOM



Ensure that all unnecessary materials are removed from the basket.



Perform precautionary wipe of all surfaces with approved wipe
****maintain wet time as stated by manufacturer ****
 For a List of approved wipes and wet times click [here](#)

INSIDE THE ROOM, ONCE ULTRASOUND COMPLETED, AFTER SANITIZING GLOVES WITH GEL, BUT BEFORE REMOVING PPE



Visually inspect the machine for any bodily fluid or tissue
 - If present, remove with wet gauze or bleach wipes



For a list of approved wipes click [here](#)

Using approved wipe, wet all surfaces of the machine including:

- the cord
- the keyboard
- the screen
- probes and cords
- the power cord
- the lid
- the wells
- gel bottles
- wipes containers

****please remember that there is a “wet time” associated with all wipes, check the manufactures recommendation****



While still in PPE, move the wet machine out of the room
****If another person is unavailable to help, use your foot to push the machine out****

AFTER EXITING THE ROOM, WHILE WEARING CLEAN PPE



Visually inspect the machine for any bodily fluid or tissue.
If present, remove with wipes

<https://www.epa.gov/pesticide-registration/list-n-disinfectants-use-against-sars-cov-2>

Using an approved wipe, wet all surfaces listed above again
**** Dry completely between the 2 required “wets” ****



Maintain wet for **required amount of time** before considering the device decontaminated
****Please check the wet times of you approved wipes and don't forget to wipe the gel bottles and wipes container as well****